BRAUN

ExactFit™ 3

ExactFit™ 5

BP 6000
BP 6100
BP 6200

GB English.....	4
BG български.....	14
CZ Česky.....	24
DE Deutsch.....	34
DK Dansk.....	44
ES Español.....	54
FI Suomi.....	64
FR Français.....	74
GR Ελληνικά.....	84
HR Hrvatski.....	94
HU Magyar.....	104
IT Italiano.....	114

NL Nederlands.....	124
NO Norsk.....	134
PL Polski.....	144
PT Português.....	154
RO Română.....	164
RU Русский.....	174
SE Svenska.....	184
SI Slovenščina.....	194
SK Slovenčina.....	204
TR Türkçe.....	214
SA اللغة العربية.....	235

Product description

BP6000
BP6100
BP6200

Product description


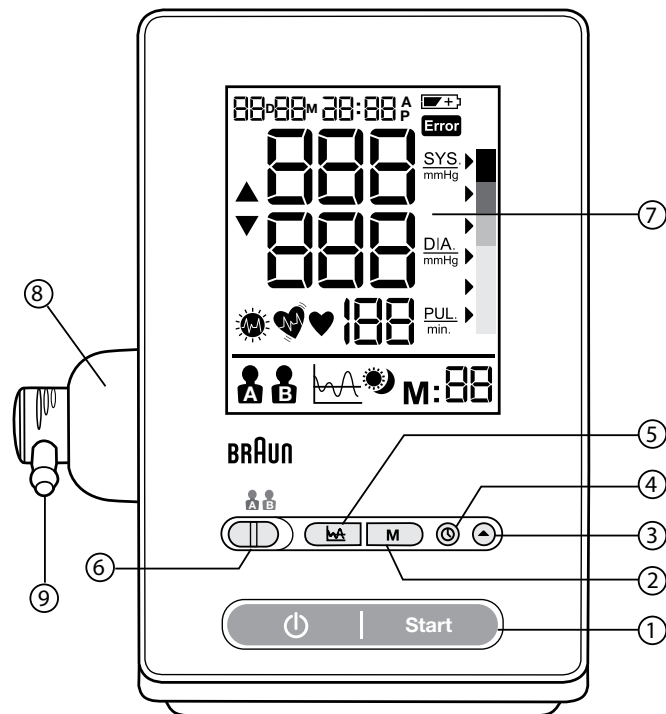
Fig. 1

Intended use of Braun ExactFit 3 and ExactFit 5

Braun upper arm blood pressure monitor has been developed for accurate and comfortable blood pressure measurements. The measuring accuracy of Braun's upper arm blood pressure monitor was tested at the time of manufacture and was proven by clinical research in accordance with ESH.

What you should know about blood pressure

Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. Also, it may vary in a short period of time. Therefore, readings from successive measurements can fluctuate.

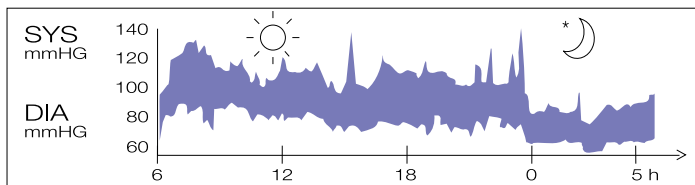Blood pressure readings taken from a healthy 31-year-old male, measured at 5-minute intervals
 Blood pressure measured in a doctor's office only provides a momentary value. Repeated measurements at home better reflect one's actual blood pressure values under everyday conditions. Moreover, many people have a different blood pressure when they measure at home, because they tend to be more relaxed than when in the doctor's office. Regular blood pressure measurements taken at home can provide your doctor with valuable information on your normal blood pressure values under actual «everyday» conditions.

The World Health Organisation (WHO) has set up the following standard blood pressure values when measured at resting pulse.

Blood pressure (mmHg)	Normal values	Mild hypertension	Severe hypertension
SYS = systole (upper value)	up to 140	140-180	over 180
DIA = diastole (lower value)	up to 90	90-110	over 110

- To ensure accurate measuring results, carefully read the complete use instructions.
- This product is intended for household use only. Keep product and batteries away from children.
- People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves, since deviations in blood pressure values may occur in such cases.
- If you are under medical treatment or taking any medication, please consult your doctor first.
- The use of this blood pressure monitor is not intended as a substitute for consultation with your doctor.

Product description (See page 2-3, Fig. 1)

1. Start button
2. Memory button **M**
3. Date / time adjust button
4. Set button
5. Average button
6. User A / B switch
7. LCD display
8. Hose port
9. Connector
10. Arm cuff
11. Air hose
12. Battery compartment cover

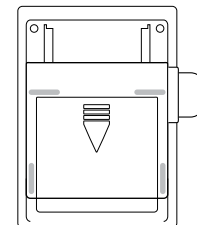
Fig. 2

Inserting batteries (See Fig. 2-3)

- Remove the battery compartment cover at the bottom of the unit and insert 4 AA LR6 alkaline batteries with correct polarity (see symbol in the battery compartment).
- Note: always re-set date and time after replacing new batteries to make sure the measurement results are stored with correct date and time.

Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.

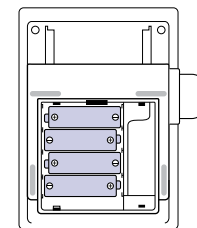
Fig. 3

Key rules for accurate blood pressure measurement

- Always take readings at the same time of day, ideally in the morning and evening, under the same conditions.
- Do not measure within 30 minutes after smoking or consuming coffee or tea.
- Take off wrist watch and jewelry before fitting the cuff on the measuring arm.
- While taking a measurement, sit down, relax, keep still, and do not move or speak.
- Wrap the cuff snugly around your arm. The cuff must be at heart level.
- Do not vibrate the unit during measurement, or the proper measurement will not be achieved.
- Perform measurement quietly in a relaxed position.
- Sit in a chair with your feet flat on the floor.
- Do not wrap the cuff over jacket or sweater sleeve or measurement cannot be done.
- Remove tight-fitting clothing from your left arm.
- Do not in any way twist the arm cuff.

- Do not inflate the monitor's cuff when it is not wrapped around the arm.
- Do not attempt to disassemble or change any parts of the monitor including the cuff.
- Do not drop the product or put it through strong impact.
- The device is not supposed to be used if your arm has any wound or injury.

Choosing the Right Cuff

For accurate measurement, it is important to choose the correct size cuff which best fits your upper arm. Choose the cuff size according to your arm circumference and make sure the bottom of the cuff is 2~3cm above your elbow.

- Small/Medium Cuff = 22~32cm arm circumference
- Large/XLarge Cuff = 32~42cm arm circumference

Applying the arm cuff

1. Slide the end of arm cuff furthest from the tube through the metal ring to a loop. The smooth cloth should be on the inside of the cuff.

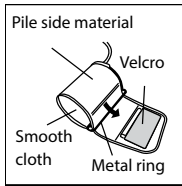
Fig. 4

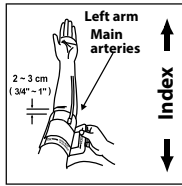
Fig. 5

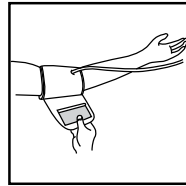
Fig. 6

2. If the cuff is located correctly, the velcro will be on the outside of the cuff and the metal ring will not touch the skin (Fig. 4).
3. Put your left arm through the cuff loop. The bottom of the cuff should be approximately (2~3 cm) above elbow. The tube should lie over the brachial artery on the inside of the arm (Fig. 5).

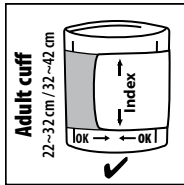
Fig. 7


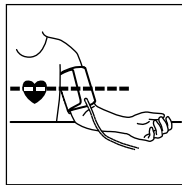
Fig. 8

4. Pull the cuff so that the top and bottom edges are tightened around your arm (Fig. 6).
5. When the cuff is positioned properly, press the Velcro firmly against the pile side of the cuff.
6. This cuff is suitable for use if the <<index>> mark falls within the <<ok range>> marked by two arrows when the cuff is tightened around your arm (Fig. 7).
7. Sit on a chair and place your arm on the table so that the cuff is at the same level as your heart (Fig. 8).

Display BP6000 / BP6100 and BP6200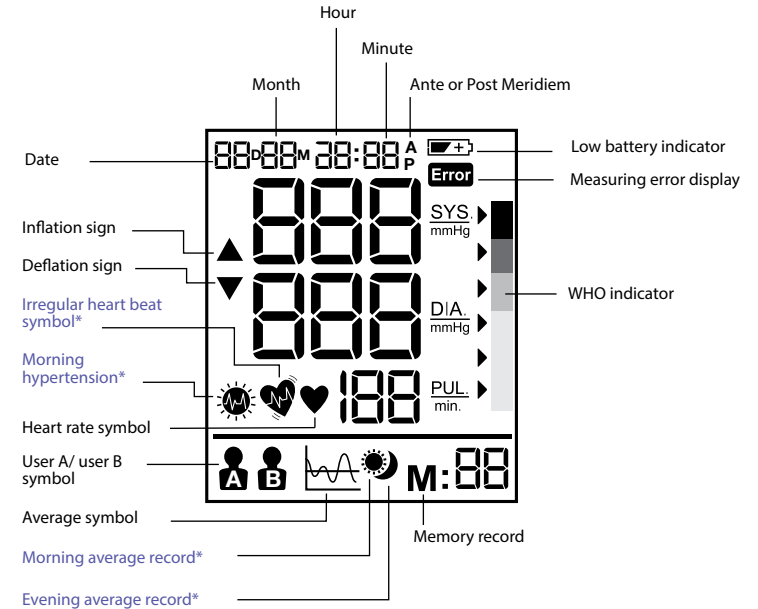

* For BP6200 only


Note: On BP6200, the backlight goes on when the unit is turned on and it will remain on until the device will be switched off.

Select mode

How to select user A /user B

- Make sure the Product is in power off mode.
- Slide the user switch to user A or user B, the current user mode will flash on the LCD display.


The WHO/ESH indicator to evaluate blood pressure data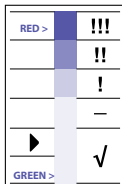

This device has a blood pressure level indicator established according to WHO and the European Hypertension Society Guidelines (ESH) in 2007. For every measurement displayed on the screen, the cursor will indicate the blood pressure level with the corresponding color code, from green to red. You can use this classification daily to guide you to understand your blood pressure level. If you are really concerned by the classification level, you should consult your doctor.

Setting month, date and time

- Switch off the device for the date/time setting.
- User press set button (1) (4) to start the year setting, then the "year" will blink display, user can press adjust (3) to adjust year by increase step "1".
- User press set button (1) (4) to start the month setting, then the "month" will blink display, user can press adjust (3) to adjust month by increase step "1".
- User press set button (1) (4) to start the day setting, then the "day" will blink display, user can press adjust (3) to adjust day by increase step "1".
- User press set button (1) (4) to start the hour setting, then the "hour" will blink display, user can press adjust (3) to adjust hour by increase step "1".
- User press set button (1) (4) to start the minute setting, then the "minute" will blink display, user can press adjust (3) to adjust minute by increase step "1".
- User press set button (1) (4) to end all the date/time setting, all the blink are stopped.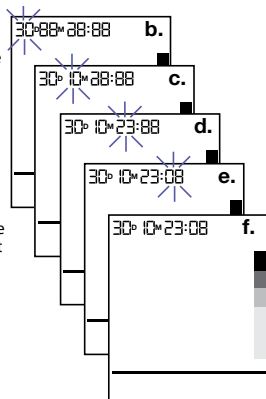

Note: Holding the adjust button will scroll the value.

Taking a measurement

Wrap the cuff around the arm (see "applying the arm cuff" section above).

1. Sit upright on the chair to have a correct posture.
2. Press and release start button (1), date/time and current user will be displayed.
3. Set user A/B switch (6) to A for user A or B for user B, LCD display will show user A or B symbol.
4. Press and release the start button (1), all icons on display will be shown for 2 seconds. The device will adjust to zero automatically. The measuring blood pressure symbol will then flash on the display and the air pressure will automatically pump up to certain pressure level and start measurement.

Do not move or talk in the midst of taking blood pressure measurement.

5. After the air pressure has increased, the pulse is detected, the heart rate symbol (♥) will start flashing.

6. LCD Display will show the results and WHO indicator arrow after measurement.

After taking blood pressure measurement, turn off the device by pressing the start button (1) or automatically after 1 minute.

Memory function

Your blood pressure monitor can store the latest 40 readings for both users each for the BP6000, 50 readings for both users each for the BP6100 and 60 readings for both users each for the BP6200.


Storing measurement data

After each blood pressure measurement, the Systolic pressure, diastolic pressure, pulse rate and the time & date of specific day will be automatically stored. Memory #01 is always the most recent one. Once the memory is full, the oldest values will be overwritten.

Press memory button M (2) to review the stored data. The last memory data (sys/dia/pul) with measurement date/time, IHB (for BP6200 only) and WHO indicator will show on LCD. Press memory button M (2) again to show the previous data. Be aware that the correct user A or B is chosen.

Average function for BP6000

Press the average button (5) to show last 3 readings average on LCD. Press average button again to clear LCD for 0.5 seconds, then results will be shown again.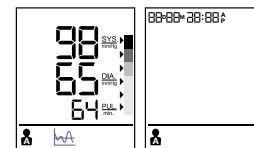


Average function for BP6100

Press the average button (5) to show the full day average of past 7 days on LCD. Press average button again to clear LCD for 0.5 seconds, then results will be shown again.


Average function for BP6200

Press the average button (5) to show the full day average of past 7 days on LCD. Press the average button second to show the morning average of past 7 days on LCD. If the result is with morning hypertension, the morning hypertension icon (☀) will be shown. Press the average button third time to show the evening average of past 7 days on LCD. Press the average button fourth time to show the full day average of past 7 days again.

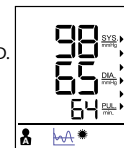Erasing data

Press and hold memory button **M** (2) for more than 5 seconds, LCD will blink display “dEL ALL” (if the “slide switch is in User A side, the user A icon will show”) or “dEL ALL” (if slide switch is in User B side, the user B icon will show).

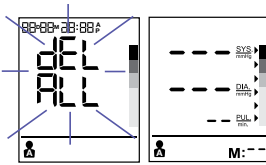Press memory button **M** (2) again, LCD will display “--” to mean all the stored data of corresponding user have been deleted.

Irregular heart beat detector (for BP6200 only)

The appearance of this symbol  signifies that a certain pulse irregularity was detected during the measurement. Talking, moving, shaking or an irregular pulse during the measurement can result in the appearance of this icon. Usually this is not a cause for concern, however if the symbol appears often, we recommend you seek medical advice. The device does not replace a cardiac examination, but serve to detect pulse irregularities at an early stage.

Low battery indicator

When the low battery indicator  flashes on the display, it means the battery is low and the four batteries need to be replaced with alkaline LR6 (AA) batteries.

** After replacing batteries, BPM will go into time-setting mode automatically and show the last measurement time on screen. Please set the current date / time before taking the next measurement to get correct average result.

Storage and cleaning

- Always keep the unit in the carrying case after its use.
- Do not put the item directly under the sunlight, in high temperature or humid and dusty places.
- Do not store in extremely low (less than -20°C) or high (more than 60°C) temperature.
- Use a piece of cloth with water or mild cleansing agent to clean the case and then use a piece of dry cloth to wipe it dry. Use a piece of dry cloth to wipe the cuff when it is dirty.
- Do not use any strong cleansers to clean it.
- When the unit is not to be used for a long time, remove the batteries. (Batteries may leak or cause harm).

Calibration

This device was designed and manufactured for a long service life, however it is generally recommended to have the device inspected once a year to ensure correct function and accuracy. Please contact the authorised service centre located in your country. Note: The calibration checking is not a free service. Please contact the Authorized Service Centre to get a quotation before you send out the product.

This device is not intended as a substitute for regular check-ups by your doctor, please continue to visit your doctor on a regular basis for a professional reading.

Production date is given by the LOT located in the back of the device. The first 3 numeric digits after the LOT No represents the day of the year of manufacture. The next 2 numeric digits represent the last two numbers of the calendar year of manufacture and the letters at the end designate the manufacturer of the product. E.g. LOT 15612VAL this product was made on the day 156, year 2012 at manufacturer identifier VAL.

What to do if

Problem	Reason	Solution
Heart rate symbol 	Appears in the measurement condition and flashes when pulse is detected.	• Measurement in progress, remain quiet.
Low battery indicator 	Appears when the battery voltage is excessively low or the positions of the batteries are incorrect.	• Replace all four batteries with new ones. Insert the batteries in the correct positions. Be aware of +/- positions.
Measuring error 	Appears when the accurate blood pressure and pulse could not be obtained.	<ul style="list-style-type: none"> • Press “start/stop” button again and remeasure. • Check if cuff is wrapped according to instructions. • Check that there are no kinks in tube. • Check palm if exerting effort. • Check if talking or moving during measurement. • Check if posture is correct.
E1 shows	The cuff is not secure	• Refasten the cuff and then measure again.
E2 shows	The cuff is very tight	• Refasten the cuff and then measure again.
E3 shows	Over-pressure	• Relax for a moment and then measure again.
E4 shows	The monitor detected motion while measuring	• Movement can affect the measurement. Relax for a moment and then measure again.
Exxx shows	Measure incorrectly	<ul style="list-style-type: none"> • Relax for a moment and then measure again. • Retake the measurement. If the problem persists, contact the retailer or our customer service department for further assistance. Refer to the warranty for contact information and return instructions.

Specifications

Method of measurement	Oscillometric
Model number	BP6000, BP6100, BP6200
Range of measurement	Pressure 0~300 mmHg Pulse 40~199 beats/minute
Accuracy	Pressure +/- 3 mmHg Pulse +/- 5 % Max.
Inflation	Deluxe automatic
Display	Liquid crystal display – systolic, diastolic, pulse rate Backlight Display for BP6200
Sets of memory	BP6000: 40 sets per user BP6100: 50 sets per user BP6200: 60 sets per user
Cuff size	Small cuff = 22-32cm arm circumference Large cuff = 32-42cm arm circumference
Operating temperature	+10 °C ~ + 40 °C, less than 85 % R.H.
Storage temperature	-20 °C ~ +60 °C, less than 85 % R.H.
Unit weight	Approximately 500gr (without batteries)
Power supply	Alkaline battery: 4 x AA (LR6)
Battery life	300 times measurement
Auto power off	Whenever not used for 1 minute
Accessories	4 batteries, 2 arm cuffs with tube, instruction manual, pouch, travel bag

IMPORTANT

Read the operating instructions.

Classification:

- Internally powered equipment
- Type B equipment
- IP22
- Not suitable for use in the presence of flammable anesthetic mixture with air, oxygen or nitrous oxide
- Continuous operation with short-time loading

Operating temperature

Storage temperature

This product conforms to the provisions of the EC directive 93/42/EEC (Medical Device Directive). This device conforms to the following standards:

- EN 60601-1: 2006 + AC:2010: - General requirements for basic safety and essential performance
- EN 60601-1-2:2007 - Electromagnetic compatibility requirements and tests
- EN 60601-1-11:2010 - Requirements for medical electrical equipment and medical electrical systems used in the home healthcare environment
- EN 1060-1:1995 + A2:2009 – Non-invasive sphygmomanometers - general requirements
- EN 1060-3:1997 + A2:2009 - Non-invasive sphygmomanometers - Supplementary requirements for electro-mechanical blood pressure measuring systems.
- EN 1060-4:2004 - Non-invasive sphygmomanometers - Test procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers.

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC. For detailed description of EMC requirements please contact an authorized local Service Centre (see package insert).

Portable and mobile RF communications equipment can affect medical electrical equipment.

Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.

Guarantee

See warranty statement inside the Guarantee/Consumer Card/Authorized Service Centers booklet.

If for any reason you have to return your device to the point of sales or authorized Service Center, please fully complete the Consumer Card (provided with the device) and insert it inside of the box before you return your device.

The Consumer Card can also be found in Kaz Europe website (www.hot-europe.com/customer-care/repair-services/) and you can download it in the case that you lose the one provided with the device.

The LOT and SN of your device are printed on the rating label in the back of the product.

UK only:

This guarantee in no way affects your rights under statutory law.

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under a license to the 'Braun' trademark.
'Braun' is a registered trademark of Braun GmbH,